

# CLARITY™

## Ultrasound Gel

Medium Viscosity

Conductive, Salt & Alcohol Free, Water-Soluble Contact Medium



CLARITY Ultrasound Gel was developed as a coupling medium to facilitate transmission of the ultrasound energy from the machine head to the tissues. Good contact leads to better image quality. CLARITY Ultrasound Gel is fluid, so it fills all available spaces yet remains viscous enough so that it stays in place.

CLARITY Ultrasound Gel stays put on the probe and its “bubble free” formulation consistently delivers superior image quality, as determined in field tests of the product. CLARITY Ultrasound Gel is available in gallons and convenient 16 oz. snap-cap bottles, which can be refilled.

### FEATURES AND BENEFITS

- Highly conductive, clear (bubble free) transmission.
- Long lasting, low absorption — no need to reapply as often.
- Smooth consistency spreads easily without running or drying prematurely.
- Non-greasy, non-staining, water-soluble clear formulation.
- Contains no salt, alcohol or fragrance.
- Non-corrosive — will not damage equipment.

1 GALLON — REORDER NO: 59008

16 OZ. BOTTLE — REORDER NO: 59007

NDC 51072-055-01 NDC 51072-055-16

**Aurora Pharmaceutical, LLC**

NORTHFIELD, MINNESOTA 55057

888-215-1256 • [www.aurorapharmaceutical.com](http://www.aurorapharmaceutical.com)

6/2012 IN 50-1185 IN 50-1184 REV 00

ss000009



MANUFACTURED IN THE USA



# CLARITY™


## Ultrasound Gel

### APPLICATION

Ultrasound gel is used to avoid intense sound reflections caused by air pockets at the borderline between the ultrasonic probe and skin. The best ultrasonic media is free of air bubbles to ensure perfect sound transmission. Liquids like water and alcohols are suitable media, however, they are inappropriate with regard to their volatility and low viscosity. CLARITY Ultrasound Gel was developed with fluidity and viscosity in mind.

**DIRECTIONS:**  
Apply gel to treatment area. After use, remove with tissue or towel. Clean equipment after each use.

Store at 15°–30° C (59°–86° F).

TAKE TIME  OBSERVE LABEL DIRECTIONS

REORDER NO: 59008

MANUFACTURED BY:  
Aurora Pharmaceutical, LLC  
NORTHFIELD, MINNESOTA 55057  
888-215-1256  
www.aurorapharmaceutical.com  
IN 50-1185 REV 00


NDC 51072-055-01

# CLARITY™

## Ultrasound Gel


Medium Viscosity  
Conductive, Salt & Alcohol Free,  
Water-Soluble Contact Medium


For Animal Use Only  
For External Use Only  
1 Gallon (3.79 L)



**DIRECCIONES:**  
Aplique el gel al área de tratamiento. Después de su uso, remueva con un pañuelo o toalla. Limpie el equipo después de cada uso.

Almacene a 15°–30° C (59°–86° F).

TÓMESE EL TIEMPO  SIGA LAS INDICACIONES DE LA ETIQUETA

MANUFACTURED IN THE USA 

3 51072 05501 7

### ABOUT AURORA PHARMACEUTICAL

Aurora is devoted to improved formulations and delivery systems that are consumer friendly and which utilize state-of-the-art ingredient “green” technologies. We are committed to manufacturing products with top-quality ingredients that increase the stability, shelf life and efficacy of our products while minimizing harmful ingredients and environmental contamination.